



FetaLink Archive functions

Archive Search


- To search for any patient's archived trace, select *Archive Search* icon located on FetaLink toolbar in *Census View*.

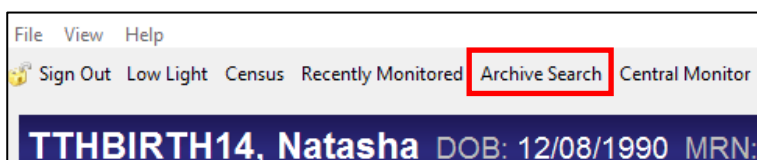


- Select *Patient Search*.

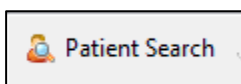

- The *Patient Search* window will display

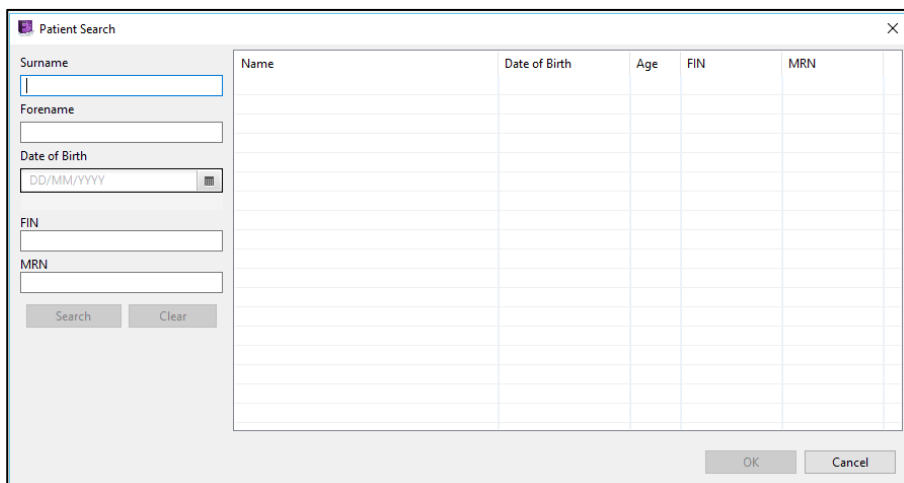


- Enter demographic details, *select your patient* and click *OK*.

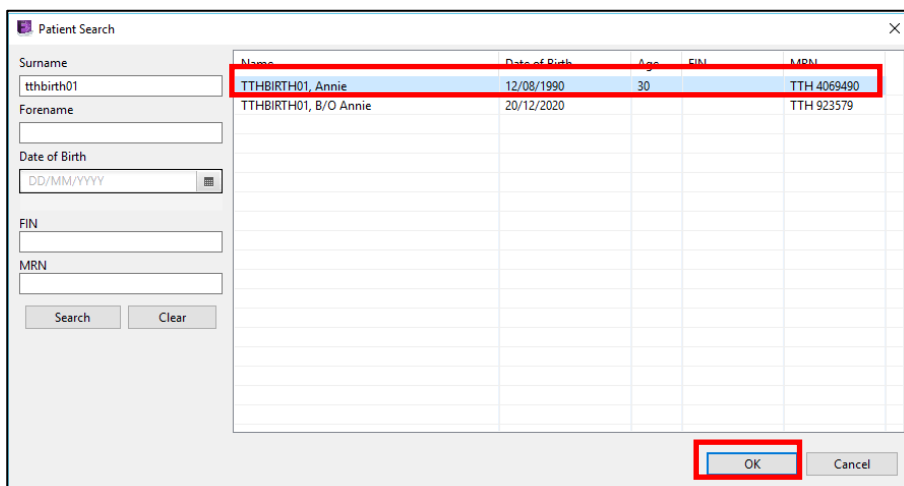

MAT - FetaLink Archive functions, v1.0, MAT###, 18/12/2020

FetaLink Archive functions


- The previously archived episodes and annotation summaries for the selected patient will display.

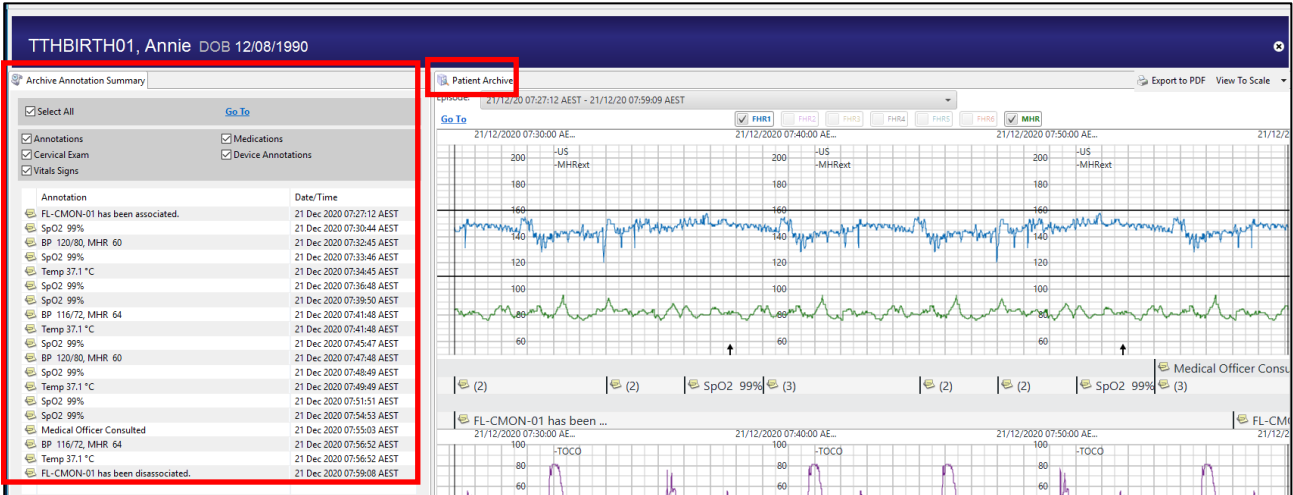


Patient Archive function

For a current patient that you are associated to:

- To search for your current patient's archived traces or monitoring episodes select *Patient Archive icon* on FetaLink toolbar in the *Location View*.

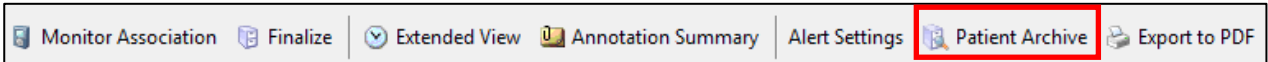

- FetaLink will search for the patient's archived trace's which will display in a drop-down box.

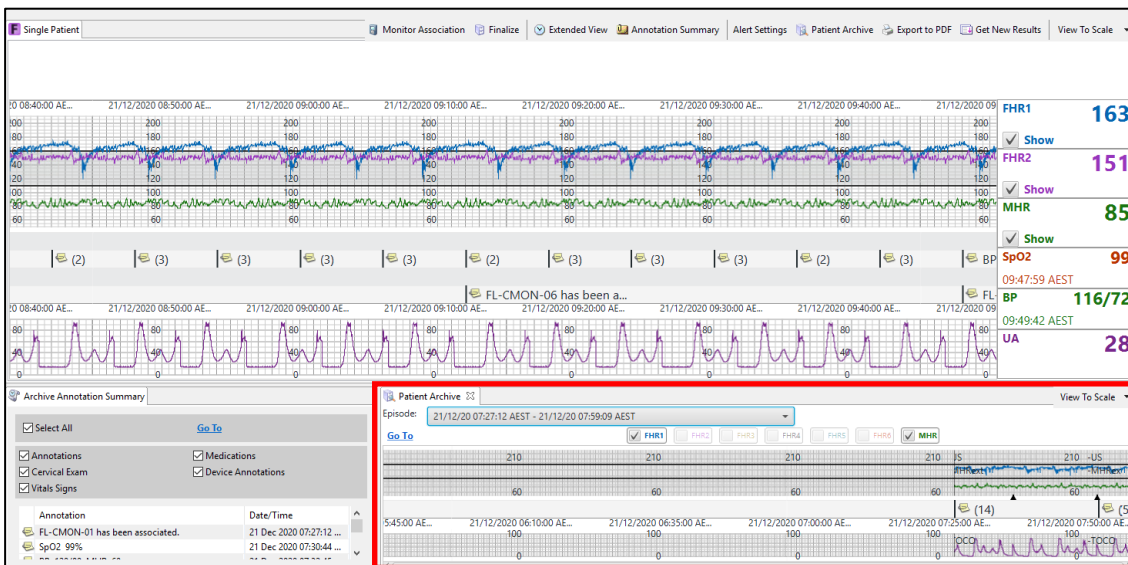

MAT - FetaLink Archive functions, v1.0, MAT###, 18/12/2020



FetaLink Archive functions

3. *Select* the appropriate patient archived trace and it will open on the bottom of the screen and the *Archive Annotation Summary* will display on the left.

