



# FetaLink Modifying association time


1. To modify the time at association, go to the *Locations view* to perform this action.
2. Select *Monitor Association*.

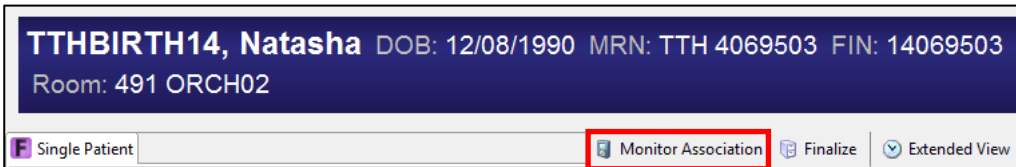

3. The *Monitor Association* window will open. *Associate* the patient to the monitor then select *Modify* to perform Association Start time change.

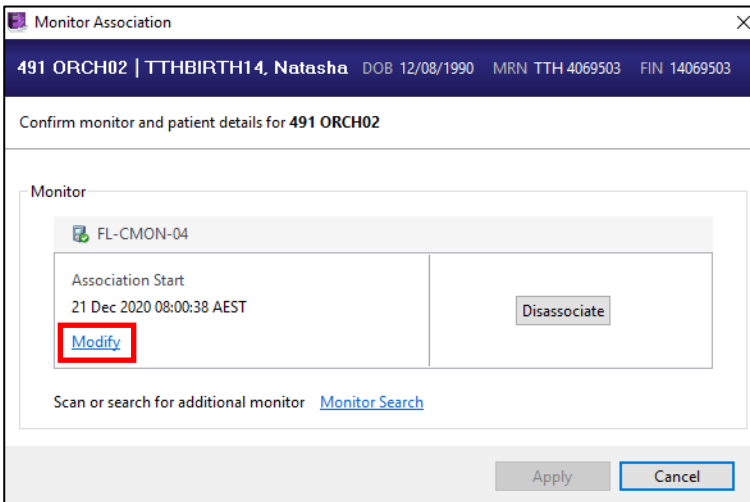


4. Update the *time field* with the correct association time, changing the *date* as applicable.

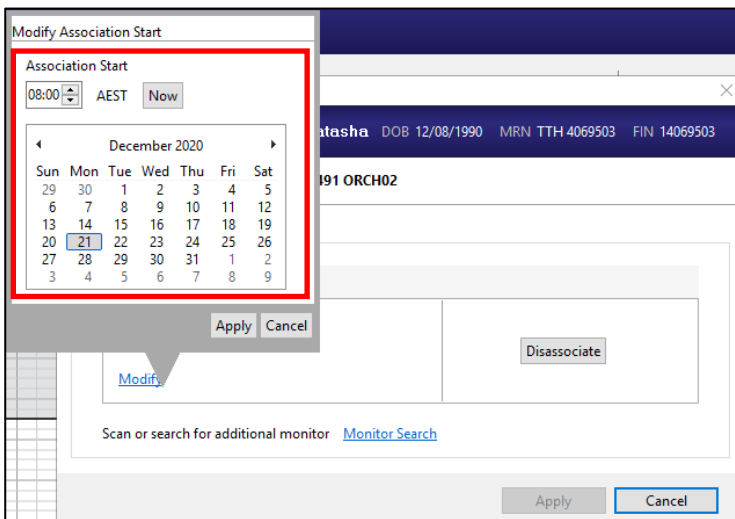

# FetaLink Modifying association time

- To *modify the time*, either type the correct time in the box or use the arrow buttons and then click *Apply*.

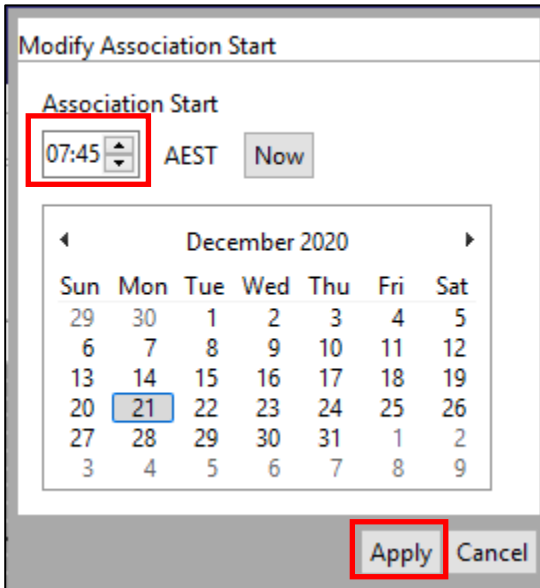

Modify Association Start

Association Start


07:45 AEST Now

December 2020

Sun	Mon	Tue	Wed	Thu	Fri	Sat
29	30	1	2	3	4	5
6	7	8	9	10	11	12
13	14	15	16	17	18	19
20	21	22	23	24	25	26
27	28	29	30	31	1	2
3	4	5	6	7	8	9

Apply Cancel

- The new associated time will display and then click *Apply* to complete. The date and time can be back dated up to 24 hours.

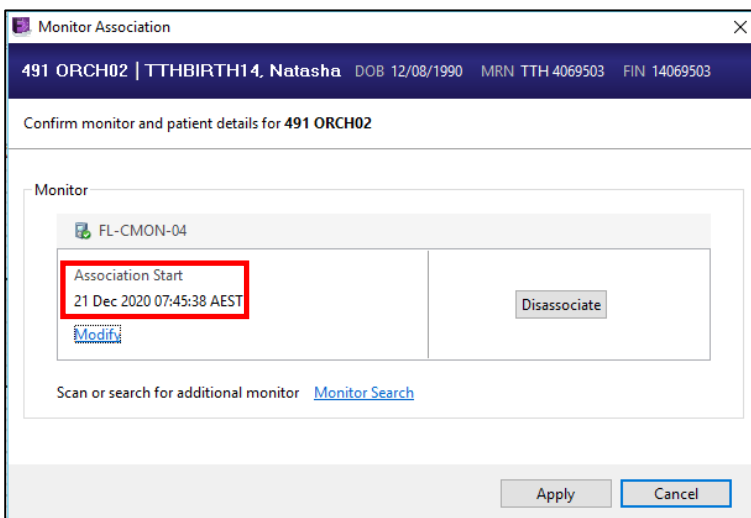

Monitor Association

491 ORCH02 | TTHBIRTH14, Natasha DOB 12/08/1990 MRN TTH 4069503 FIN 14069503

Confirm monitor and patient details for 491 ORCH02

Monitor

FL-CMON-04

Association Start  
21 Dec 2020 07:45:38 AEST

Disassociate

Modify

Scan or search for additional monitor [Monitor Search](#)

Apply Cancel