





FetaLink Toolbars and Icons

Census View Toolbar

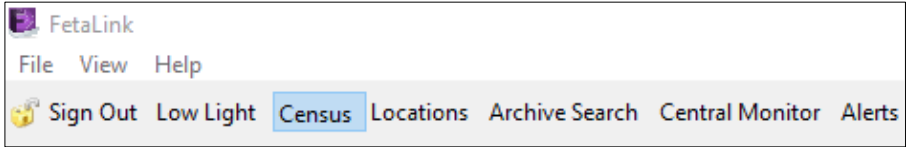









Button	Label	Action
	Sign In	Click to sign in and unlock FetaLink.
	Sign Out	Click to sign out of FetaLink.
	Census View	Opens the Census view, which displays a list of patients, with information about those patients and specified set of locations.
	Locations View	Opens the Monitored Locations view.
	Archive Search	Opens the Patient Archive for any patient.
	Central Monitor	Opens the Central Monitor view.
	Alerts (#)	Opens the Alerts view. The number in (#) is the number of patients on which alerts are currently active.

Location View Toolbar

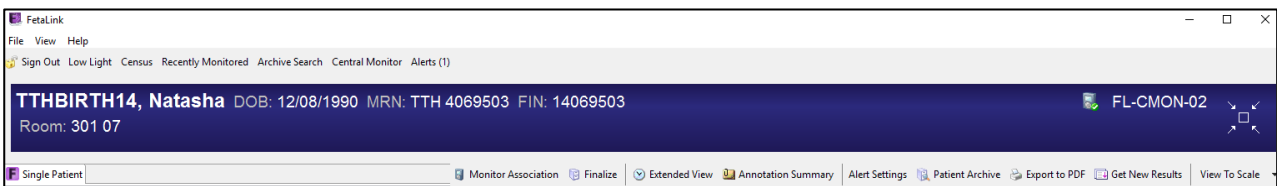



MAT - FetaLink Toolbars and Icons, v1.0, MAT###, 18/12/2020

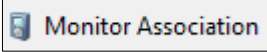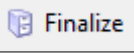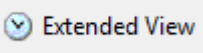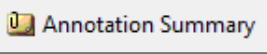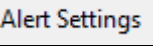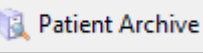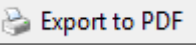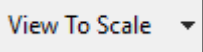








FetaLink Toolbars and Icons

Button	Label	Action
 Monitor Association	Monitor Association	Click in Location view to disassociate the monitor.
 Finalize	Finalize	Click in Location view to finalise the episode.
 Extended View	Extended View	Opens the Extended view, which allows you to view the entire monitoring episode in Single Patient view.
 Annotation Summary	Annotation Summary	Opens the Annotation Summary for the patient in context in the Monitored Locations view.
 Alert Settings	Alert Settings	Click to change alert settings for patient.
 Patient Archive	Patient Archive	Archive CTG traces from the same patient are available.
 Export to PDF	Export to PDF	Export to PDF for review or education.
 View To Scale ▾	View to Scale	This shows 20 minutes of CTG trace but can be compressed to 60 minutes.

FetaLink Icons

Icon	Action
	Click to minimize or maximize a monitor location.
	Double click to edit an annotation.

FetaLink Toolbars and Icons

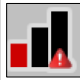	Click to annotate an alarm.
	Click to acknowledge an alarm.
	This indicates an alert has been centralised for this patient i.e. FHR alarm parameters have been alerted.
	In Census view for devices associated to a location that is not currently connected to the network.
	In Census view for devices that are connected to the network.
	In Census view when multiple devices are associated to a location and all are connected.
	In Census view when multiple devices are associated to a location and all are disconnected.
	In Census view when multiple devices are associated to a location and some are connected, and some are disconnected.
	This indicates that the monitor location has a low signal quality mode.

Border Colour

The banner bar at the top of the CTG trace will be blue when all systems are working. If the banner changes colour this is due to the following alerts.



FetaLink Toolbars and Icons

Border Colour Precedence

In the event of multiple types of alerts occurring at the same time, the border colour precedence is as follows:

1. Alert (Red)	
2. Time Out of Sync (Orange)	
3. Device Disconnected (Yellow)	
4. Low Signal Quality (Grey)	