


FetaLink Transfer patient to another location


The patient is transferred in FetaLink when their physical location changes, such as moving them from ANDAC to Birth Suite, or Birth Suite to theatre, or Birth Suite to Maternity ward.

1. In the *Census view* click on the *patient's name*

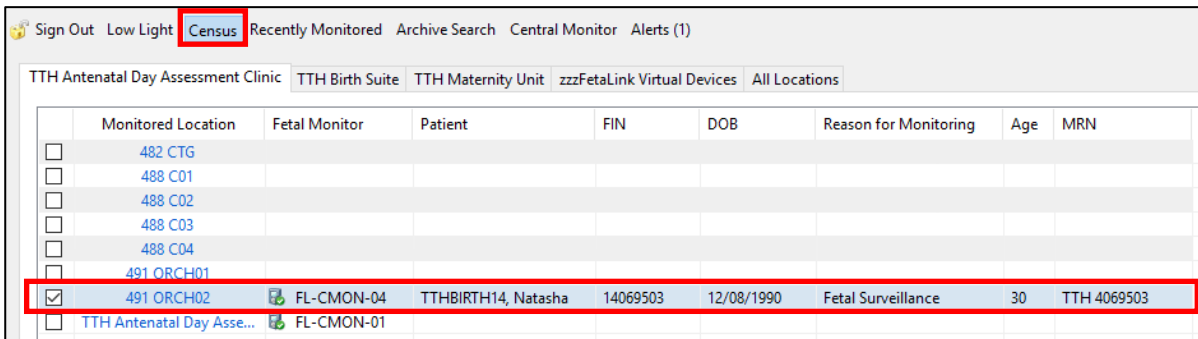

Monitored Location	Fetal Monitor	Patient	FIN	DOB	Reason for Monitoring	Age	MRN
<input type="checkbox"/> 482 CTG							
<input type="checkbox"/> 488 C01							
<input type="checkbox"/> 488 C02							
<input type="checkbox"/> 488 C03							
<input type="checkbox"/> 488 C04							
<input type="checkbox"/> 491 ORCH01							
<input checked="" type="checkbox"/> 491 ORCH02	FL-CMON-04	TTHBIRTH14, Natasha	14069503	12/08/1990	Fetal Surveillance	30	TTH 4069503
<input type="checkbox"/> TTH Antenatal Day Asse...	FL-CMON-01						

2. The transfer window will appear. You need to select the *appropriate transfer option* – Patient and all fetal monitors or Patient only.

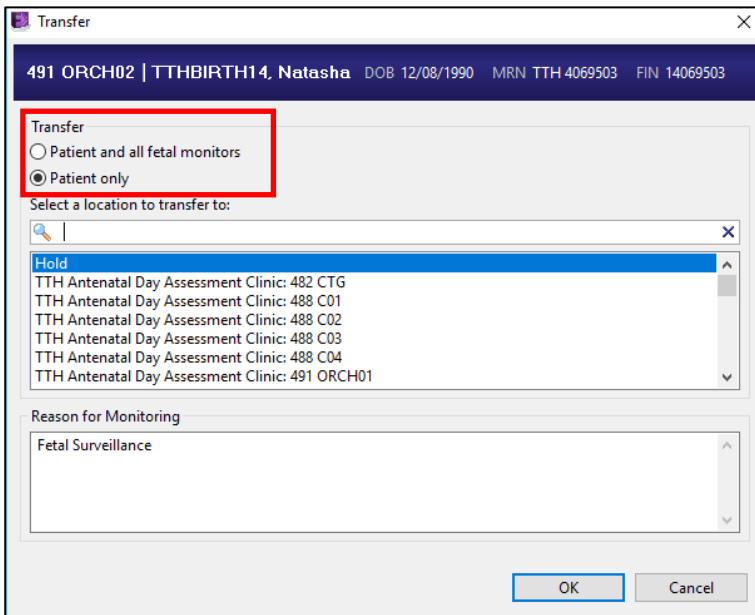

Transfer

491 ORCH02 | TTHBIRTH14, Natasha DOB 12/08/1990 MRN TTH 4069503 FIN 14069503

Transfer

Patient and all fetal monitors

Patient only

Select a location to transfer to:

Hold

- TTH Antenatal Day Assessment Clinic: 482 CTG
- TTH Antenatal Day Assessment Clinic: 488 C01
- TTH Antenatal Day Assessment Clinic: 488 C02
- TTH Antenatal Day Assessment Clinic: 488 C03
- TTH Antenatal Day Assessment Clinic: 488 C04
- TTH Antenatal Day Assessment Clinic: 491 ORCH01

Reason for Monitoring

Fetal Surveillance

OK Cancel



FetaLink Transfer patient to another location

3. Select the *patient new location* and update the *Reason for Monitoring* as required.
4. Click *OK* to complete the transfer.

Transfer

491 ORCH02 | TTHBIRTH14, Natasha DOB 12/08/1990 MRN TTH 4069503 FIN 14069503

Transfer

Patient and all fetal monitors

Patient only

Select a location to transfer to:

- TTH Antenatal Day Assessment Clinic: 488 C02
- TTH Antenatal Day Assessment Clinic: 488 C03
- TTH Antenatal Day Assessment Clinic: 488 C04
- TTH Antenatal Day Assessment Clinic: 491 ORCH01
- TTH Birth Suite: 301 07**
- TTH Birth Suite: 305 A02
- TTH Birth Suite: 507 A01

Reason for Monitoring

in labour

OK Cancel

5. The patient has now been transferred to the new location in FetaLink and has been disassociated from the CTG in the previous room.
6. To associate the patient to the new CTG, click on the *new location*.

Sign Out Low Light Census Recently Monitored Archive Search Central Monitor Alerts (1)

TTH Antenatal Day Assessment Clinic TTH Birth Suite TTH Maternity Unit zzzFetaLink Virtual Devices All Locations

Monitored Location	Fetal Monitor	Patient	FIN	DOB	Reason for Monitoring	Age	MRN
301 07		TTHBIRTH14, Natasha	14069503	12/08/1990	in labour	30	TTH 4069503
505 A02							
507 A01							
525 A03							
539 06							
541 05							
543 04							
545 03							
547 02							
549 01							
TTH Birth Suite	FL-CMON-02						
TTH Birth Suite	FL-CMON-06						
TTH Birth Suite	FL-CMON-08						
TTH Birth Suite	FL-CMON-25						



FetaLink Transfer patient to another location

7. *Scan the barcode* on the new CTG to associate.



8. Select *Apply* to complete the association.

Monitor Association

301 07 | TTHBIRTH14, Natasha DOB 12/08/1990 MRN TTH 4069503 FIN 14069503

Confirm monitor and patient details for 301 07

Monitor

FL-CMON-02

Association Start
Today, Now
[Modify](#)

Scan or search for additional monitor [Monitor Search](#)

Apply Cancel

It will now appear in the *Census view*.

Transfer from new location

1. In the *Census view*, click on the *location* that the patient will or has been transferred to.
2. The *Locations view* will open with no patient or CTG associated to this location.

No patients or fetal monitors are associated to this location.



FetaLink Transfer patient to another location

3. *Scan the patients bar code.* The transfer dialog opens with the options to Transfer the Patient ONLY or Transfer the Patient AND Fetal Monitoring Device.



4. Select the *appropriate transfer* and click *Apply*.
5. The patient is now transferred to their new location and disassociated from the fetal monitor in the previous location if applicable.

TTHBIRTH14, Natasha DOB: 12/08/1990 MRN: TTH 4069503
Room: 301 07
F Single Patient

6. Re-associate the patient to the monitor required by *scanning the CTG barcode*.



7. The *Monitor Association* will open. Select *Apply* to complete the new association.



FetaLink Transfer patient to another location

Monitor Association

301 07 | TTHBIRTH14, Natasha DOB 12/08/1990 MRN TTH 4069503 FIN 14069503

Confirm monitor and patient details for 301 07

Monitor

FL-CMON-02

Association Start
Today, Now
[Modify](#)

Scan or search for additional monitor [Monitor Search](#)

Apply Cancel